



CAN HYDRATION BE AN INDICATOR FOR PERFORMANCE?

A STATISTICAL ANALYSIS OF THE CORRELATION OF
PRE-COMPETITION HYDRATION LEVELS IN ATHLETES

I. Introduction & Background

Tanita Corporation is considered a world leader in precision electronic scales. Since 1923 scales have been the cornerstone of the global, family-owned company based in Tokyo, Japan. With a focus on health and wellness, Tanita manufactures a wide variety of scales ranging from gram kitchen scales, to pediatric scales for infants, to bariatric scales weighing patients up to 1,000 pounds. Tanita is the scale of choice for the NCAA, USA Wrestling, Jenny Craig, Weight Watchers as well as physicians and hospitals worldwide.

In 1992, Tanita developed the world's first foot-to-foot body composition analyzer. Using a technology called Bioelectrical Impedance Analysis (BIA), Tanita's body composition analyzers send a safe, low electrical signal (approximately 50 kHz, 500 Micro Amp) up one leg and down the other, carried by the water and fluids that are contained within the body to measure impedance. Muscle is normally approximately 50-70% hydrated whereas fat is only 10-15% hydrated. This impedance information is then used to calculate multiple readings including the amount of lean and fat tissue within the body, bone mass and hydration.

The process has been independently vetted as being highly consistent and repeatable alongside Dual Energy X-ray Absorptiometry (DEXA) which is widely considered the gold-standard in body composition. Tanita's FDA cleared foot-to-foot process, provides an easy to use, yet clinically accurate solution, and makes body composition assessments accessible in both the professional and medical environment as well as at home. Since its inception, Tanita has sold over 20 million body composition analyzers worldwide.

In 2004, Tanita identified a demand within the fitness industry and partnered with one of the most recognizable brands in all of fitness - IRONMAN®. Organized by the World Triathlon Corporation (WTC), and with over 30 years of history, IRONMAN® is regarded as the most honored and prestigious triathlon in the world. An IRONMAN® Triathlon consists of a 2.4-mile swim, 112-mile bike and a 26.2-mile run. What started as challenge among friends in Hawaii in 1978 has grown to more than 52 IRONMAN® branded events, covering six continents and with more than 100,000 athletes competing globally each year. IRONMAN® has become the international standard of athletic excellence and symbolizes best-in-class authenticity, performance and success.

Since the Tanita and IRONMAN® union, triathletes all over the world have utilized Tanita's body composition analyzers to monitor their weight, body fat percentage, muscle mass as well as to help calculate their sweat loss during workouts. Even before Tanita and IRONMAN® joined forces, Tanita had always addressed the importance of proper hydration to the general public.

Considering the grueling and unforgiving demand and stress put on the body during an IRONMAN® competition, Tanita suspected that a triathlete's hydration could be vital to their performance, and may even provide an indicator as to how well they will be able to compete in an IRONMAN® Triathlon. Is there a direct correlation between pre-competition hydration and performance in an IRONMAN® Triathlon?

II. Method of Data Collection

During athlete registration at the 2008, 2009 and 2010 Ford IRONMAN® Louisville, Tanita received the permission from IRONMAN® to obtain hydration readings from its participants. After the athletes had received their bib number, they were directed to one of six body composition assessment stations where a Tanita trained assessor was positioned. Six, fully calibrated, Tanita BF-522W Body Fat/Body Water Analyzers were used during the assessment process. Once at the assessment station, the athletes were asked their bib number, age and height. A reference chart converting centimeters to feet and inches were positioned at each station to accommodate international competitors unfamiliar with their imperial measurement. The assessor would then input the athletes age, height and gender into the analyzer. After removing their shoes and socks, the athlete would then stand on the device barefoot to complete the analysis. The assessor then recorded both the athlete's bib number and hydration level. Over the three years, assessments were taken on 5,740 of the competitors.

After the event, IRONMAN® provided to Tanita a detailed spreadsheet with all of the competitors' information including their bib number and completion time. The bib number recorded by the assessor during registration was then referenced and matched with the race result data and each hydration level was assigned to its appropriate athlete. The results for men and women were then separated and the athletes who did not finish the race were given a time of 18.0 hours.

III. Results and Interpretations

Over the three years, 4,426 male participants and 1,314 female participants were tested. Figure 3.1 displays the male hydration levels and the average corresponding finish times in hours.

| FIGURE 3.1 | MALE |
|--------------------|------------------|
| <i>HYDRATION %</i> | <i>AVG. TIME</i> |
| ≥68% | 11.96 |
| 66-67% | 12.54 |
| 64-65% | 12.87 |
| 62-63% | 12.88 |
| 60-61% | 13.41 |
| 58-59% | 13.92 |
| 56-57% | 14.67 |
| 54-55% | 15.25 |
| 52-53% | 15.97 |
| 50-51% | 15.99 |


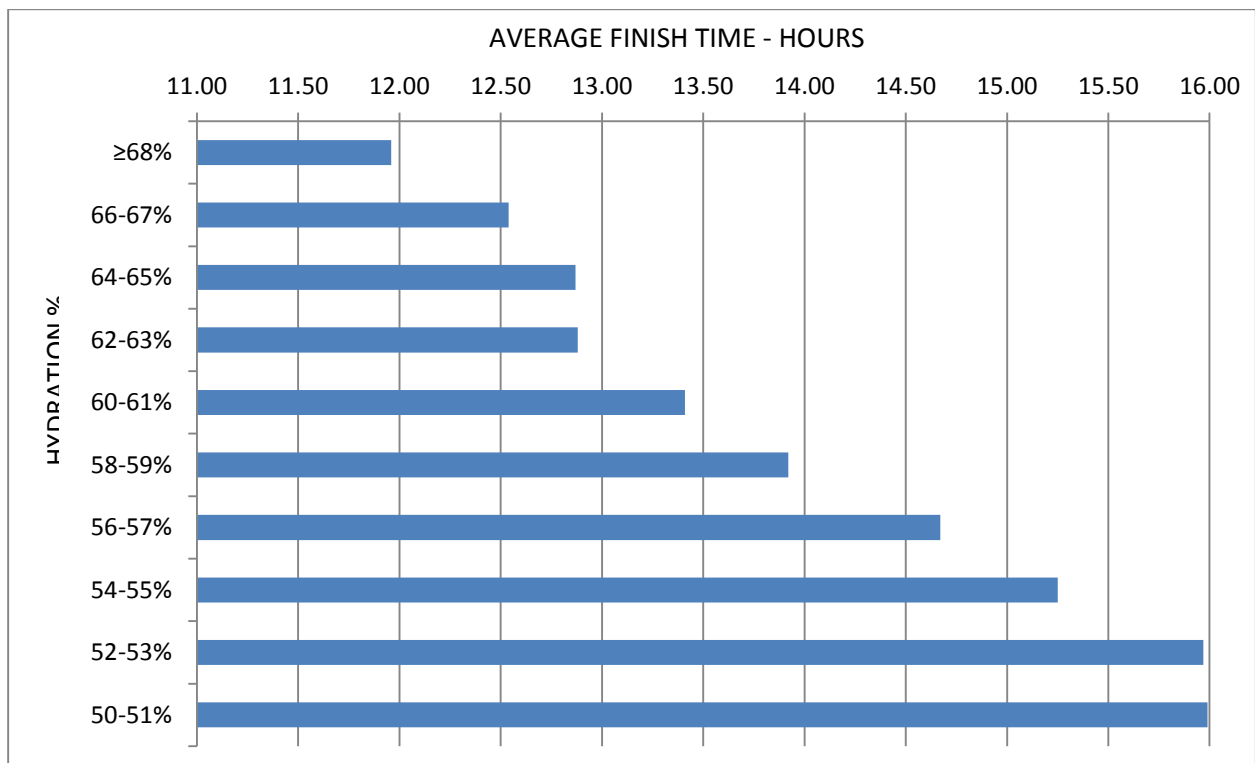


Figure 3.2 displays the female hydration levels and the average corresponding finish times in hours.

| FIGURE 3.2 | FEMALE |
|--------------------|------------------|
| <i>HYDRATION %</i> | <i>AVG. TIME</i> |
| ≥63% | 13.09 |
| 61-62% | 13.27 |
| 59-60% | 13.64 |
| 57-58% | 13.71 |
| 55-56% | 13.94 |
| 53-54% | 14.49 |
| 51-52% | 14.99 |
| 49-50% | 15.68 |
| 47-48% | 16.23 |
| 45-46% | 16.45 |

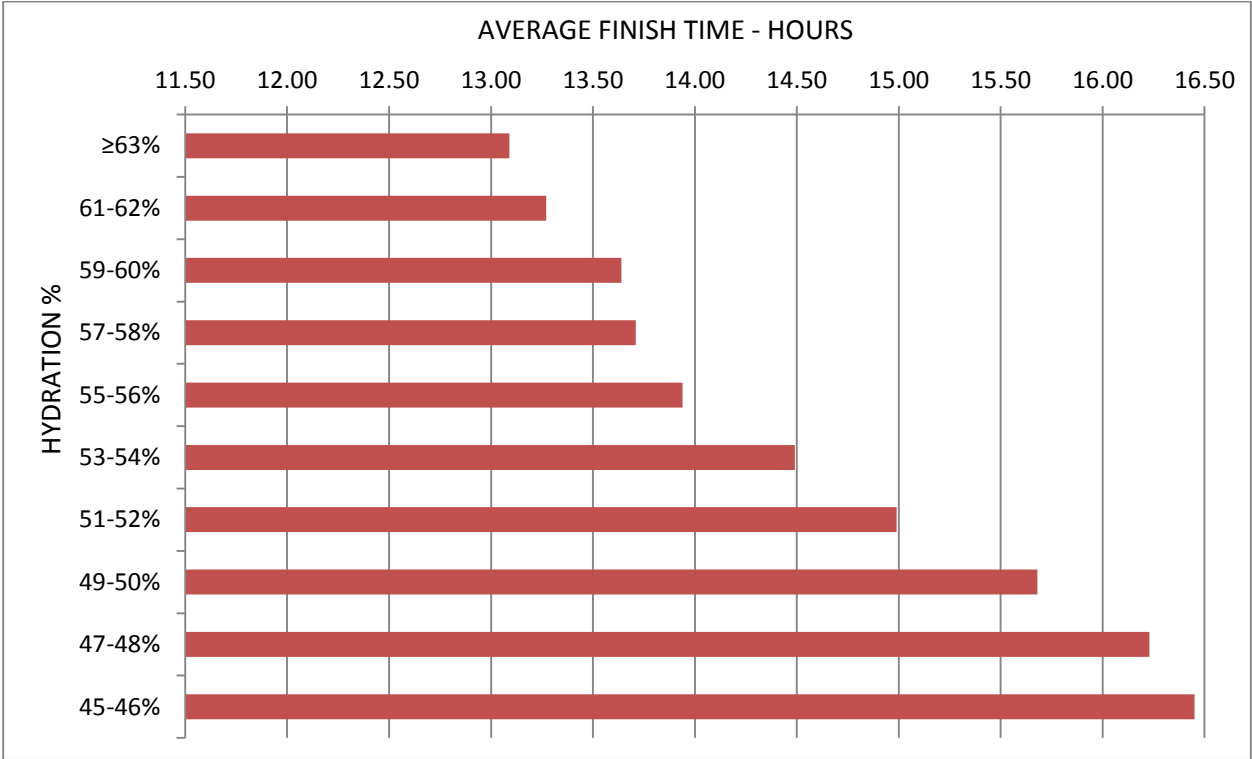

The average hydration level of men that were able to finish had a hydration level of 60%. The average hydration level of women that finished had a hydration level of 55.7%.

Of the 2,093 male participants that had a pre-competition hydration level $\geq 60\%$, 1,945 were able to finish (93%) and had an average finish time of 12.76 hours. 1,860 of the 2,106 (88%) male participants with a pre-competition hydration level $< 60\%$ and $\geq 55\%$ finished the race with an average time of 13.82 hours. Only 153 out of 227 (67%) of the male participants with a pre-competition hydration level $< 55\%$ were able to finish the race, and those that did finish had an average finish time of 14.58 hours. See Figure 3.3.


| FIGURE 3.3 | MALE | FINISHERS |
|--------------------|-----------------|------------------|
| <i>HYDRATION %</i> | <i>% FINISH</i> | <i>AVG. TIME</i> |
| $\geq 60\%$ | 93% | 12.76 |
| 55-59% | 88% | 13.82 |
| 50-54% | 67% | 14.58 |

Of the 747 female participants that had a pre-competition hydration level $\geq 55\%$, 691 were able to finish (93%) and had an average finish time of 13.41 hours. 447 of the 499 (90%) female participants with a pre-competition hydration level $< 55\%$ and $\geq 50\%$ finished the race with an average time of 14.37 hours. Only 45 out of 68 (66%) of the female participants with a pre-competition hydration level $< 50\%$ were able to finish the race, and those that did finish had an average time of 15.22 hours. See Figure 3.4.

| FIGURE 3.4 | FEMALE | FINISHERS |
|--------------------|-----------------|------------------|
| <i>HYDRATION %</i> | <i>% FINISH</i> | <i>AVG. TIME</i> |
| $\geq 55\%$ | 93% | 13.41 |
| 50-54% | 90% | 14.37 |
| 45-49% | 66% | 15.22 |


When we take all of the male data points and perform a regression analysis, we see with 95% confidence that for every 1% increase in the men’s hydration level, their mean time decreased by .2662 hours (16 minutes). With this information, we are also able to predict in 95% confidence the male finish time based on hydration. See Figure 3.5.

| FIGURE 3.5 | MALE |
|--------------------|-----------------------|
| <i>HYDRATION %</i> | <i>PREDICTED TIME</i> |
| 70 | 11.10 |
| 69 | 11.37 |
| 68 | 11.63 |
| 67 | 11.90 |
| 66 | 12.16 |
| 65 | 12.43 |
| 64 | 12.70 |
| 63 | 12.96 |
| 62 | 13.23 |
| 61 | 13.50 |
| 60 | 13.76 |
| 59 | 14.03 |
| 58 | 14.29 |
| 57 | 14.56 |
| 56 | 14.83 |
| 55 | 15.09 |
| 54 | 15.36 |
| 53 | 15.63 |
| 52 | 15.89 |
| 51 | 16.16 |
| 50 | 16.42 |

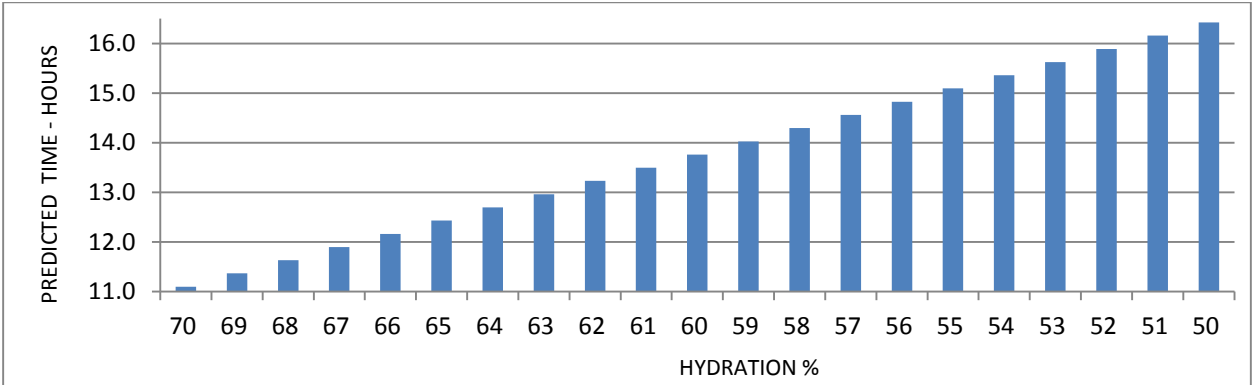

Likewise, when we take all of the female data points, we see with 95% confidence that for every 1% increase in hydration level, female's mean time decreased by .2036 hours (12 minutes). Figure 3.6 shows in 95% confidence females finish time base on hydration.

| FIGURE 3.6 | FEMALE |
|--------------------|-----------------------|
| <i>HYDRATION %</i> | <i>PREDICTED TIME</i> |
| 65 | 12.32 |
| 64 | 12.52 |
| 63 | 12.73 |
| 62 | 12.93 |
| 61 | 13.13 |
| 60 | 13.34 |
| 59 | 13.54 |
| 58 | 13.75 |
| 57 | 13.95 |
| 56 | 14.15 |
| 55 | 14.36 |
| 54 | 14.56 |
| 53 | 14.76 |
| 52 | 14.97 |
| 51 | 15.17 |
| 50 | 15.37 |
| 49 | 15.58 |
| 48 | 15.78 |
| 47 | 15.98 |
| 46 | 16.19 |
| 45 | 16.39 |

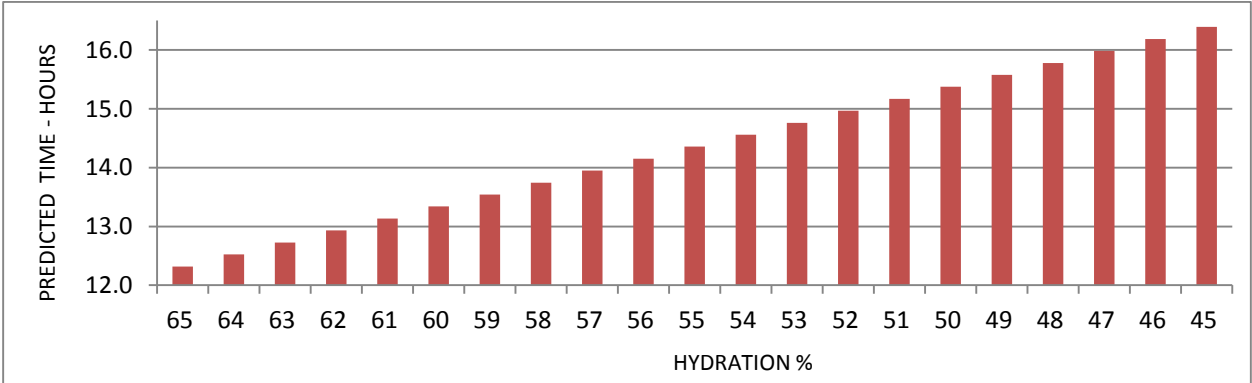

IV. Summary and Conclusion

For years, coaches, trainers and athletes alike have recognized the importance of hydration before, during and after athletic competitions. It is essential in cooling the body through sweat and helps prevent the muscles from cramping. Without proper fluid intake, it can lead to lightheadedness, nausea, vomiting and even death. This investigation has allowed us to take a closer look to see if there is a relationship between an individual's hydration level and how well they are able to perform in an IRONMAN® Triathlon.

Figures 3.1 and 3.2 represent a strong indication that a relationship exists between hydration % and time. Individuals with the higher hydration levels, on average, had better times. Men with a hydration level $\geq 68\%$, on average, had a finish time of 11.96 hours. One can then easily see that as hydration levels decreased, mean time increased. At the bottom of our results, the men with a hydration level between 50 and 51% had an average finish time of 15.99 hours. That is more than a 4 hour difference between the most and least hydrated male participants. Women with a hydration level $\geq 63\%$, on average, had a finish time of 13.09 hours. Likewise, women with a hydration level between 45 and 46% had an average finish time of 16.45 hours. That represents nearly a 3.5 hour difference between the most and least hydrated female participants.

Figures 3.3 and 3.4 reveal that certain thresholds of hydration levels maybe extremely vital as to one's ability to finish an IRONMAN® race. 93% of male participants with a hydration level $\geq 60\%$ were able to successfully finish the race. However, only 67% of men with a hydration level $< 55\%$ were able to finish. Similarly, 93% of female participants with a hydration level $\geq 55\%$ finished the race. Only 66% of the female participants were able to finish whom had a hydration level $< 50\%$.

By performing a regression analysis, we were able to see that not only is there a relationship between hydration level and time, but that with every 1% increase in hydration, the men's mean time decreased by .2662 hours (16 minutes) and .2036 hours (12 minutes) for the women. With this information, we are able to take things a step further and predict within 95% confidence, what the mean time will be based on hydration levels.

The instant a triathlete takes the plunge into the water to begin their 140.6 mile journey; many questions weigh upon their mind. Will my wetsuit tear? What happens if I get a flat tire? Will it rain? Will it be windy? Can I do this? With so many factors out of their hands playing roles into how well they will do, it appears that through this paper, there is at least one thing they can control that will help decipher their day.

What is my hydration?

IRONMAN® and the "M-DOT" logo are registered trademarks of World Triathlon Corporation. Official Product of the IRONMAN® TRIATHALON. Used here by permission.

These are the conclusions drawn by Tanita. Tanita leaves the reader to draw their own conclusions based on the data contained herein.

This data was not collected under strict clinical conditions; rather it represents real-world results spanning three years of testing.

Hydration on its own is not a guarantee of success. It is however a strong marker that athletes should use to gauge their condition and potential performance.